



AED Monthly Checklist

6 Nov 2024

Complete

Score	15 / 15 (100%)	Flagged items	0	Actions	0
Location of AED	Oak View, Main Street, Fenton, Newark England NG23 5DE United Kingdom (53.04738833158851, -0.6886448542712633)				
AED Serial No.	0000137073				
Checked on	6 Nov 2024 09:33 GMT				
Checked by	Graham Doney				

AED Monthly Checklist	15 / 15 (100%)
Is the AED secure in its container?	Yes
Check AED is on standby mode - green flashing light	<input checked="" type="checkbox"/>
If AED is not on standby mode, verify AED operation	
Verify AED Operation	4 / 4 (100%)
Open AED lid / Turn on AED	<input checked="" type="checkbox"/>
Wait for status indicator to change from RED to GREEN after five seconds	<input checked="" type="checkbox"/>
Close AED lid / Turn off AED	<input checked="" type="checkbox"/>
Wait for status indicator to change from RED to GREEN after five seconds	<input checked="" type="checkbox"/>
Are the electrode pads in the AED container?	Yes
Check expiration date of electrode pads	<input checked="" type="checkbox"/>
Electrode Pads Expiration Date:	25 Nov 2024
Is there an extra set of electrode pads?	Yes
Expire May 2025	
Check expiration date of AED battery	<input checked="" type="checkbox"/>
The AED battery is expired if 4 years have passed since its installation.	
AED Battery Installation Date:	10 Jul 2023
Check PPE/Ready Kit	4 / 4 (100%)
Mask	<input checked="" type="checkbox"/>
Scissors	<input checked="" type="checkbox"/>
Razor	<input checked="" type="checkbox"/>
Gloves	<input checked="" type="checkbox"/>
Test AED container tamper alarm	<input checked="" type="checkbox"/>

IMPORTANT Maintenance

If electrode pads were expired, did you replace them?

N/A

Input the date the electrode pads were replaced:

If AED battery was expired, did you replace it?

N/A

Input the date the AED battery was replaced:

If the AED container tamper alarm was not working, did you change its battery?

N/A

Signature



Graham Doney
6 Nov 2024 10:27 GMT